

# OTTER


PORTABLE POOL LIFT



nibotechnics

**The Otter portable pool lift is a flexible and convenient system for anyone who needs a safe way of getting into and out of a swimming pool.**

The Otter can be used by most pools; whether public swimming pools, pools at hotels, pools at other holiday resorts or even by your own private pool. Whenever it is time for exercises or for a swim, the Otter can be wheeled up to the pool and installed by the poolside without trouble. It only takes a little help from you to lower the swimmer into the water.

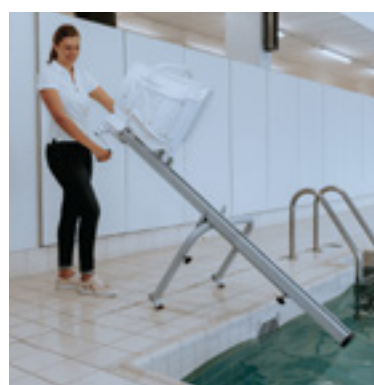

Fasten the 'T' hooks into the sockets. They are quick to fit and remove from the installation points, which are barely noticeable when not in use.


Bring the lift from where it is stored to the pool. The wheels make this easy.

Locate the Otters' claw feet under the 'T' hooks and lower the unit into place. It is then ready for immediate use.



**EASY TO OPERATE.  
SAFE AND COMFORTABLE FOR THE SWIMMER.**

Fold down the armrest. This gives the swimmer security.

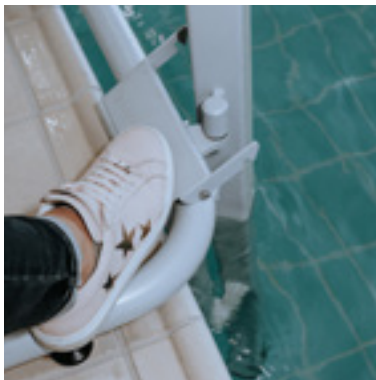

Depress the locking plate to unlock the lift. It locks automatically when in a safe position; that is over the deck or over the water.

Unlock the lift and swing the chair over the water. This is a smooth, easy operation.

Turn the handle and lower the swimmer into the water. It is smooth, safe and takes very little effort.

Fold back the armrest. With the water supporting some of his weight, the swimmer can ease himself out of the chair on his own, or with some assistance.

The fold-back armrest also allows for safe and comfortable side transfers from a wheelchair onto the seat.

## PRODUCT SPECIFICATIONS

safe working load	160 kg (352 lbs)
length of stroke, max.	1450 mm (57")
length of stroke above water surface, min.	720 mm (28 3/8")
length of stroke below water surface, max.	730 mm (23 3/4")
total product height (mast & launch)	2085 mm (82")
"minimum" recommended space requirement	1775 mm (69 7/8")
STD Otter weight (complete)	39 kg (85 lbs)
max. base frame width	700 mm (28")

The Otter pool lift has been designed for safe and easy operation at the poolside.

The mast and base assemblies are made of stainless steel with a moulded plastic seat.

The elevating and lowering operation is mechanical. In-built into the design is a "free wheel" device which immediately stops the chair lowering should it meet an obstruction.

A locking mechanism incorporated in the base frame automatically locks the chair for safe transfer, into and out of the water.

The Otter pool lift can normally be installed at any poolside.

### **Nibotechnics nv**

Heikant 5 | 3930 Hamont-Achel | Belgium

T +32 (11) 80 20 40 | [info@nibotechnics.com](mailto:info@nibotechnics.com) | [www.nibotechnics.com](http://www.nibotechnics.com)