MEDICAL FITNESS

Lojer Pulleys

Operating and
maintenance
manual

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Ostopäivä/Date of purchase _____

Your local dealer _____

**Lojer Pulley ja Vertical pull
Operating and maintenance manual**

CE

Contents

1	Lojer Pulleys	1
1.1	Description of parts.....	1
1.2	Accessories.....	2
1.3	Symbols used on the device.....	2
2	Introduction	3
2.1	Inspection upon delivery.....	3
2.2	Before use.....	3
2.3	Installing the Mobile Pulley.....	4
2.4	Installing the Vertical Pull.....	4
3	Using the Pulley.....	5
3.1	Using the mobile stand (Mobile Pulley only).....	5
3.2	Adjusting the weights.....	5
3.3	Adjusting the draw height.....	6
3.4	Length adjustment of the rope.....	6
3.5	Changing of the drawing handle of Pulleys.....	6
4	Cleaning and disinfection	7
4.1	Metal and plastic surfaces.....	7
5	Maintenance	8
5.1	Daily measures.....	8
5.2	Biannual and annual measures.....	8
5.3	Changing the Pulley rope.....	9
5.4	Splicing the rope of the Vertical Pull.....	10
5.5	Spare parts.....	10
6	Technical information.....	11
6.1	Standards.....	11
7	Recycling	11

Read the instructions fully before using the device.

1 Lojer Pulleys

The Lojer Pulleys are to be used in medical fitness and rehabilitation under physiotherapist's supervision, together with a training plan made by the therapy professional. Training plan should include the method, accessories, weight and number of repeats.

This user manual is made for following products: Lojer Pulleys, Lojer Speed Pulleys and vertical pulls.

This document gives instructions for operating and maintaining Lojer Pulleys. Please familiarise yourself with these instructions before using the device. Use the device only as described and for the specified applications. Store these instructions in an appropriate way, making sure that the instructions are available to all possible users throughout the life of the device.

To avoid injury, follow the instructions given in this document.

1.1 Description of parts

The sections of Lojer Pulleys are shown below (Figure 1 and Figure 2).

Figure 1: Lojer Pulleys

- 1 Pulley wheel(s)
- 2 Drawing handle
- 3 Adjustment fork
- 4 Height adjustment knobs
- 5 Weight stack
- 6 Total kilograms of weight stack
- 7 Central locking pedal
- 8 Transportation wheels

Figure 2: Lojer Vertical Pull

- 1 Crossbar (gantry)
- 2 Lower adjusting wheel
- 3 Handle
- 4 Adjusting fork
- 5 Height adjustment knob

1.2 Accessories

Lojer Pulleys are designed to be used with number of accessories (Figure 3), that enable different type of exercises. Use only accessories that are defined by the manufacturer. See more at www.lojer.com or ask from your Lojer dealer.

Figure 3: Accessories

1.3 Symbols used on the device

Symbol	Description
CE	This product meet the requirements of Medical Device Directive 93/42/EEC

Warning label (squeezing hazard)

This symbol points out important safety instructions, which must be very carefully followed. By following the instructions of this manual you ensure the personal safety and property of yourself, your patients and others.

2 Introduction

2.1 Inspection upon delivery

Before the device is taken into use, check that the packaging is intact and that it has not been damaged during transportation. Please notify the transport company and the supplier of any transit damage within two (2) days of receiving the delivery.

Ensure that the delivery contains all the parts detailed in the delivery note. If there is anything missing from the delivery consignment, please contact the supplier immediately.

The device can be stored at a temperature of -10...+60 °C. The permitted humidity is 30...75 %.

2.2 Before use

Remove all the bands and the blocks around the weight stack before using the product.

The device is intended to be used in normal, dry indoor conditions. Ensure that the temperature of the room is between +10...40 °C and the humidity is within the range of 30...75 %. If there is chance that device has been exposed to temperatures below 0°C, allow it to adjust to the indoor temperature for at least 5 hours before using any of its features.

Familiarise yourself with the instructions and carry out the following before using the device:

- Make sure that all packing materials have been removed.
- Make sure that the device can freely move up and down.

Do not place anything under the device (e.g. chair, because the device has to be able move freely). Make sure that the patient's limbs do not get caught in the frame of the device.

Do not place the device under any wall structures or too close to the wall.

Do not modify the structure of the device or install parts other than those mentioned in this document.

Do not use the device or the accessory if it doesn't work properly. Contact the service

Do not push the device on to a door sill.

The physiotherapist should always supervise the training.

Inspect the Pulley before use. Do not use if the Pulley appears damaged or inoperable.

Inspect all ropes, straps and their connections. Do not use if the component is found worn or damaged.

Keep body and clothing free and clear of all moving parts. Failure to do so could result in personal injury.

Do not use dumbbells or other means to incrementally increase the weight resistance, other than those provided by the manufacturer.

Do not modify the structure of the Pulley or install parts other than those mentioned in this document on the device.

WARNING! Children or people with no experience of the device or those with restricted understanding must not use the device. Children must be supervised to ensure that they do not play with the device!

2.3 Installing the Mobile Pulley

Assemble the mobile section to the Speed Pulley (Figure 4) with supports and screws exactly to vertically position in case the delivery occurs separately.

Lift the pulley to the mobile stand

Screw M8X20 (4) both sides

Screw the back bracket on M6X20 with washers (4). Make sure the pulley is in straight line.

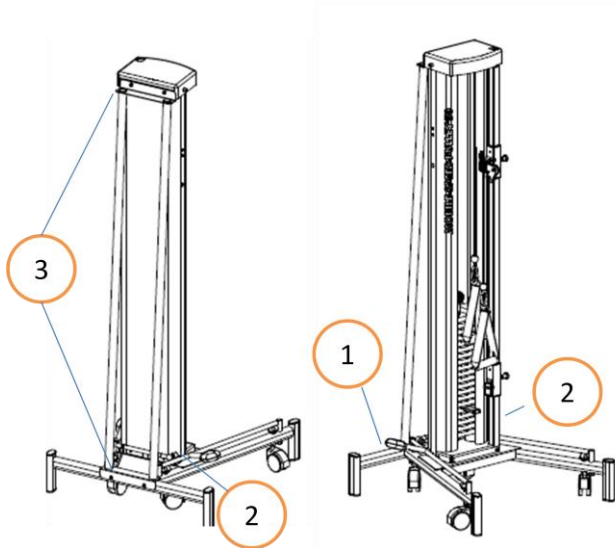
Figure 4: Installing the mobile pulley

2.4 Installing the Vertical Pull

Fix the Vertical Pull against a solid wall that can stand the utilization of the device. The Vertical Pull must be installed exactly vertically (when looked at both from the side and from the front).

Install the Vertical Pull as follows:

First compile the top part of the Vertical Pull by turning the bearer into its place and clamping the bolts. Then push the top part into its place inside the frame (Figure 5).

Attach the Vertical Pull firmly against the wall. Fix the top part (the bearer) to the wall as high as possible, the maximum height being however 70 cm from its lowest position.

Pull the rope/wire tight and make sure it functions properly.

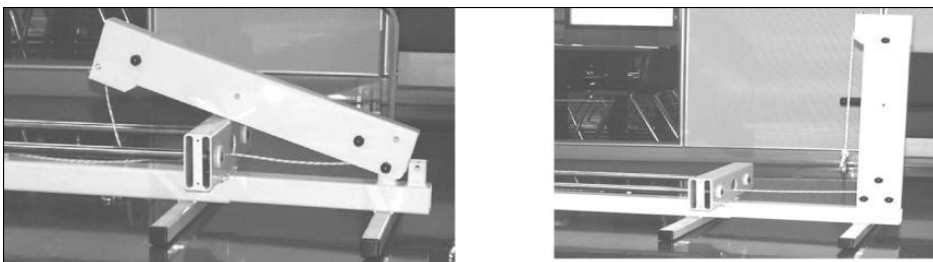
Figure 5: Installing the vertical pull

Make sure the wall is solid enough before installing the device.

3 Using the Pulley

3.1 Using the mobile stand (Mobile Pulley only)

When you are using the device be sure that the device is standing on the legs and the wheels are lifted up. If you want to move the device, press the pedal (Figure 6) downwards, and the wheels are on.

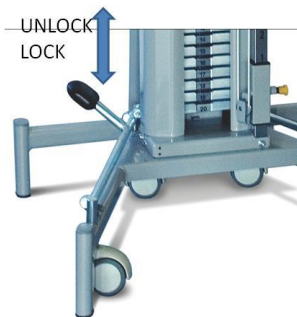
Figure 6: Locking pedal

SQUEEZING HAZARD! Make sure that nothing is or gets under the device or between the structure of the device during lifting/lowering.

3.2 Adjusting the weights

Pulley:

In the pulleys you can adjust the weights at 0,5 – 1,0 kg intervals by placing the adjusting fork in a slot under the desired weight amount. There is also available a set of extra weights (Figure 7)(3 x 0,250 kg, see: Accessories) that you can add above the weight stack when needed. Make sure that extra weights are properly attached and they cannot drop between the weight stack and casing.

Figure 7: Extra weights

Speed Pulley:

you can adjust the weights by the following: The ratio of the resistance is 1:5, the resistance coming out is 0,5 kg's per rope per plate. For instance 4 weight stack plates means 2 kg resistance per rope. By combining the ropes so that you have only one handle to pull, the resistance is double, for instance 4 weight plates – resistance 4 kg. Weights can be adjusted by placing the adjusting fork in a slot under the desired weight amount.

Vertical Pull:

You can adjust the weights at 5 kg tolerance by placing the adjusting fork in a slot under the desired weight amount.

Before using the devices, make sure that the fork is correctly positioned at the end of the slot of weight stack, and the rope and its hook are not damaged.

Make sure that mobile pulley is not on the wheels when using the device.

3.3 Adjusting the draw height

You can adjust the draw height from the adjusting bar (Figure 8) in front of the device (the clear tube with holes). You can adjust the draw height or the length of the rope by pulling the adjusting knob and at the same time moving the adjusting parts up- or downwards.

When using a Vertical Pull device with the lower adjusting wheel, do not pull the rope to the side.

Figure 8: Adjusting bar

3.4 Length adjustment of the rope

The rope length can be adjusted from the lower adjustment piece at the chromium plated bar (Figure 8).

3.5 Changing of the drawing handle of Pulleys

The handle is changed by opening the hook at the end of the rope, removing the old handle and replacing it with a new one. You may use various kinds of drawing handles in Pulleys. You get the standard triangle handles along with the Pulley (see Accessories catalog or ask your local dealer).

4 Cleaning and disinfection

Before cleaning remove all accessories.

Clean stains as soon as possible.

In order to keep the surfaces in good condition do the cleaning regularly. Do cleaning/disinfectant always between patients. Do more thorough cleaning once a month. Follow the cleaning/disinfecting instructions given by the respective facility.

4.1 Metal and plastic surfaces

Clean the metal and plastic surfaces and with a damp cloth and weak alkaline cleaning fluid. Use small brush for corners and other difficult spots. Rinse with clean water and dry carefully after cleaning. Do not use excessive fluids.

Use disinfectant (alcohol or chlorine) and follow the disinfectant manufacturer's instructions for use. Let dry by evaporation in room temperature.

Plastic surfaces (ABS, HDPE, PP) are highly resistant to chemicals. Plastic is resistant to bleaching agents (alkaline compounds), dilute organic or inorganic acids. Also solvents and cleaning agents may be used.

Plastic surfaces may get damaged if aromatic hydrocarbons (benzene and its derivatives), ketones, ethers, esters and chlorinated hydrocarbons are used. Plastic might also deteriorate if it is exposed to various chemicals at the same time.

Stainless steel surfaces are highly resistant to chemicals. Use for mild detergent solution for cleaning. Ammonia and most of the solvents can be used to remove difficult stains. Avoid chlorine based solutions.

Painted or chromed metal surfaces can be cleaned with mild detergent. They are also highly resistant to chemicals. Do not use harsh abrasive powders on these surfaces.

All surfaces must be dry before using the device.

Do not use water spray (shower, high-pressure water guns) for cleaning.

Do not clean in high temperature and air humidity.

Do not expose the device to excessive moisture which can result in liquid pooling.

Do not use solvents or petrol for cleaning. Do not use acids for cleaning.

Dry all surfaces thoroughly after cleaning or disinfection.

Disinfecting wears out the surfaces. After disinfecting clean the surfaces with clean, damp cloth. Dilute the disinfectant according to the manufacturer's instructions.

Recommendable cleaning agent: Lojer Desiplint (1:10), which is effective against bacteria without drying the upholstery material.

Repeat the cleaning procedure if necessary. (Composition of Lojer Desiplint: Chlorhexidine-digluconate 0,1 – 0,2 %, water 99,8 %.)

After disinfecting, wipe the surface with a clean, damp cloth after disinfecting.

Dried substances or substances that have penetrated for an extended period may not be able to be removed completely

5 Maintenance

Only trained person may carry out service and repair. Maintenance carried out by an authorized person may cause injury or damage to the device which the manufacturer is not responsible for.

All service and repair operations must be documented.

5.1 Daily measures

Inspect all ropes, straps and their connections.

Do not use if the component is found worn or damaged.

5.2 Biannual and annual measures

Lubricate the slide bars (25 mm barrels, in encased models inside the case) of the weight stack approximately every six months. For example, use WD-40 spray lubricant.

Check also the condition of the rope and its hook regularly and change them into new ones, when necessary.

Change the rope after 12 months of utilization. Use always original LOJER rope.

If the Pulley makes abnormal noises or does not function correctly, perform the maintenance measures without delay and/or contact the distributor's maintenance service.

5.3 Changing the Pulley rope

Splicing of the rope is shown below (Figure 9).

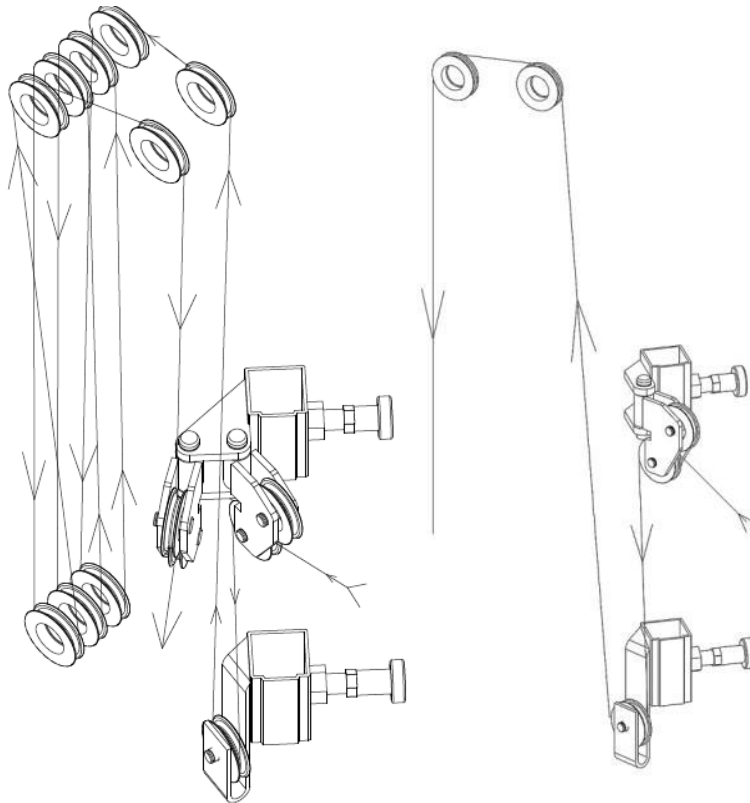
Figure 9: Splicing of the rope (Speed Pulley on the left)

- Slide the rope through the right height adjustment wheel.
- Slip it then downwards and round the lower adjustment wheel so that the rope comes out from behind of the wheel.
- Pull the rope then all the way up and slide it through the little hole and over the wheel so that the rope comes out from behind of the wheel.
- Draw the rope behind the first upper wheel, and then sequentially round every wheel (down – up – down ...) according to the pictures.
- Change the last circle so that the rope goes behind the last upper wheel and comes out from the front of the wheel.
- Slip the rope then through the left adjustment wheel.

5.4 Splicing the rope of the Vertical Pull

Splicing the rope to lower adjusting wheel:

Attach the rope to the weight stack.

Splice the rope.

Turn the adjustment roll down so, that the yellow triangle sticker is on the bottom of the roll (Figure 10a).

When changing the rope, make sure that the rope attached correctly (picture). The rope must be spliced crosswise (Figure 10b).

Figure 10: a) Triangle sticker downwards b) Locking the rope

5.5 Spare parts

To order spare parts contact the Lojer service. Before contacting the service, find out the following information from the type plate of the device.

- Name, model and the serial number of the device
- Date of purchase
- Description of the problem

Most common spare parts are shown below.

		Pulley	Speed Pulley	Vertical Pull
LM104Ä	Rope Lojer Vertical Pulley			✓
LM105Ä	Rope Lojer Speed Pulley		✓	
LM106Ä	Rope Lojer Pulley	✓		
R14544	Pulley wheel, plastic	✓	✓	
SE9275Ä	Weight stack fork Pulley, small	✓		
LL15500	Weight stack fork LMF, big		✓	✓
R2826171K	Adjusting knob GN 817, yellow	✓	✓	
R282617K	Adjusting knob GN 817, yellow, self-locking	✓	✓	

6 Technical information

Check the information also from the type plate (Figure 11).

	50kg speed pulley	14-20kg pulley	Speed pulley 80-100kg	Vertical pull 50-80-100kg
Total weight	68kg	30-36kg	154-174kg	142-172-192kg
Lenght	18 cm	18cm	46cm	82cm
Width	26cm	26cm	51cm	51cm
Height	230cm	230cm	216cm	228-298cm
Weight stack	50kg	14-20kg	80-100kg	50-80-100kg

Figure 11: Type plate

6.1 Standards

The device is in conformity with essential requirements of the Directive 93/42/EEC (Medical devices) and the corresponding Finnish National Law no. 629 (2010). The device is marked with CE marking. The device is classified as Class I medical device according to the directive.

7 Recycling

Most of the materials used in the device are recyclable. When the device is removed from usage, it should be disassembled and recycled appropriately. Recycling should be done by specialized company. Do not dispose the device in the household waste.

Remove the battery from the device. The oil from the hydraulic system should be removed and disposed appropriately. Gas springs should be unpressurised and oilfree before recycling.

Following materials should be separated before recycling:

- METALS: frame, screws, nails, springs etc.
- ENERGY WASTE (combustible waste): wood and wood-based materials.
- ELECTRIC WASTE: wires, power cords, actuators etc.
- HOUSE HOLD WASTE: plastic, upholsteries and other materials which cannot be separated further.

Contact your local disposal authority for more details of how to recycle. Follow the instructions given in local collection points.

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Lojer Oy
Putajantie 42, P.O.Box 54
FIN-38201 Sastamala, Finland

declares that

***LOJER 19100 Pulley, 19200 Speed Pulley, 19200M Mobile Pulley and
19300 Vertical Pull***

are in conformity with the following essential requirements of the Directive 93/42/EEC (Medical Devices) and the corresponding Finnish National Law no. 629 (2010).

The designing of these devices is based on the following harmonized standards:

EN ISO 14971:2009

Quality Management System: ISO 9001:2000

The products are marked with the CE marking

Sastamala, Finland 2012-09-01
LOJER OY

*Aliina Tuomenlehti
Risk management engineer*

Lojer Oy
P.O.BOX 54
Putajantie 42
FI 38201 Sastamala, FINLAND
Tel. +358 10 380 6700
Fax + 358 10 380 6702
email: firstname.lastname@lojer.com
www.lojer.com
VAT No: FI01443540
Domicile: Sastamala

Manufacturer
Lojer Oy
Putajantie 42, P.O.Box 54
FI-38201 Sastamala
Finland
Tel. +358 10 830 6700
Fax. +358 10 830 6702

www.lojer.com
service@lojer.com